



PRE-INSTALLATION SIGN-OFF


by SEDECAL

Please check the boxes indicating you have reviewed the topic and the facility is compliant. **We recommend you consult with your electrician or contractor when reviewing and completing this form.** Please call our Technical Support if you have any questions. All orders must have this form completed and returned before Sedecal can confirm installation.

X-ray system (1) normally located on centerline of the wall.

Recommended minimum room size - 9W x 8D x 8H
 (10W x 8D x 8H for Premium Vets.)
 (For smaller rooms please call for information on additional options.)

Safety Disconnect (2) is a surface mounted box that can be located either left or right of table and 5' centerline of box off floor or to state code height. Safety disconnect should have a true earth grounding wire going back to the main panel and an additional empty ground lug for 6 AWG wire. No neutral is needed for single or three phase operation.

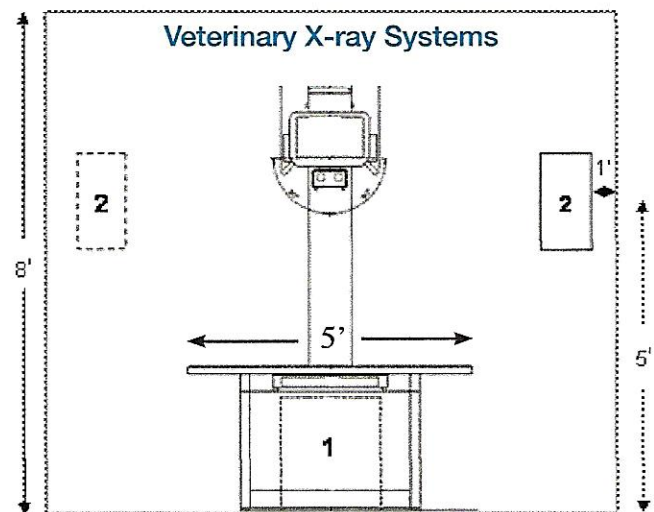

Electrical Requirements - Unless otherwise ordered, all units shipped are single phase units.
Safety disconnect and wire size requirements are as follows:

Single Phase, 80 Amp safety disconnect minimum
 (208VAC to 240VAC)

0' to 50' - 2 AWG

51' to 100' - 1/0 AWG

For runs over 100' call Sedecal USA for other x-ray generator options.



Ross Hazlett

President

p.h # (978) 597-7000

fax # (978) 597-7002

cell # (978) 808-7713

web - www.nemedrepair.com
 email - nemedrepair@gmail.com

Veterinary - Medical - Dental
X-Ray - Digital - Surgical & Patient Care Equipment
Sales - Service - Installation & Preventative Maintenance

All wire sizes stated above are for **COPPER** wire, and run lengths are from the building main electrical panel to room mounted safety disconnect.

For Technical Support, call: 847.394.6961